



STERILE INSTRUMENT INTEGRITY: READY TO USE?

| | PEEL POUCHES | WRAPPED TRAYS & RIGID CONTAINERS |
|------------------------|--|--|
| Storage | - No contamination or damage | - No contamination or damage |
| Package | - No stains, holes, or writing on paper side - Few small items packaged - Sealed properly | - No stains, dirt, holes, or damage |
| Load Label | - Visible on plastic side | - Visible |
| Instrument | - Unlocked; not ratcheted or closed - Clean, dry, and no rust, pitting, or damage - No disposable instruments or household items | - Unlocked; not ratcheted or closed - No peel pouches inside tray - No disposable instruments or household items |
| Indicators | - External & Internal: Correct color change | - External: Tape intact, stripes present - Internal: Correct color change & location |
| ID Tape | - No peeling, damage, residue, or gaps - Wrapped 1.5 times, no flat application | - No peeling, damage, residue, or gaps - Wrapped 1.5 times, no flat application |
| Double Pouch | - Inner pouch not folded; only sealed - Both from same manufacturer - Both facing same way; plastic-plastic - Allowed by manufacturer | |
| Rigid Container | | - Filter present, intact, and dry - Break away intact |

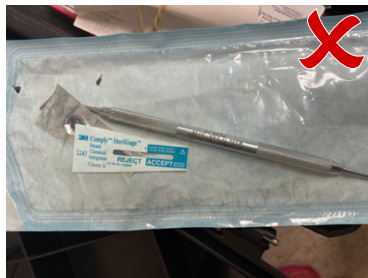

Peel Pouch with Holes

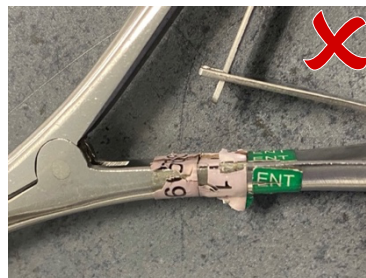


ID Tape Damaged & Peeling




Folded Inner Peel Pouch



Broken External Indicator Tape

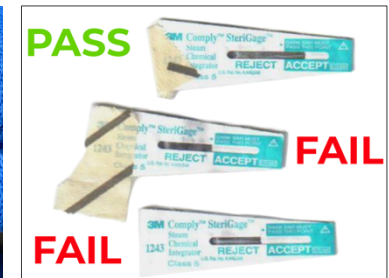

Class 5 Chemical Indicators

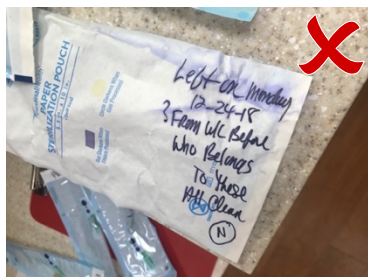

Writing on Paper Side



Double Peel Pouch: Different Manufacturers



ID Tape Flat & Damaged



ID Tape Peeling & Disposable Instrument



Intact Break-Away & Load Sticker